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Réf: 178420 / 178400

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MANUAL BREAST PUMP INSTRUCTIONS

IMPORTANT - Read the instructions carefully and keep them for the future reference.

Using a breast pump is an effective solution for:

- If you are going to be away from your baby, or if you are not able to feed the baby because of illness or prematurity, you will need to pump to prepare feeding bottles of baby.
- Unclog her breasts for better comfort in case of rising milk severe or inflamed breasts.
- Give fragile children the benefit of their excess milk by donating milk to the lactarium in its region.
- Initially the milk production will be slight but it's important to continue to pump to stimulate the milk production. As milk production becomes more established, more milk will be obtained through pumping.

The breast pump is light, ergonomic and comfortable thanks to its soft silicone pad for a very gentle use. Completely removable to ensure good hygiene, it is easy to use. Its suction force is similar to baby's natural rhythm.

The breast milk is the best food adapted of the metabolism of baby, who digests it more easily and more quickly than artificial milk.

INSTRUCTIONS:

- Before first use : disassemble, clean and sterilize the breast pump, feeding bottle, the teat and accessories (see the part «cleaning the breast pump»).
- Wash your hand with soapy water before touching your breast or the pump.
- When you are pumping, choose a quiet place where you are able to relax.
- Apply warmth to your breast before pumping. (warm cloth or warm shower).
- Massage the breast in a circular motion and gently rub the nipple just like you would do for self breast exam.
- Place the horn to your breast with the opening centred over your nipple. Wetting the inside of the horn helps the nipple to move freely on the inside of the horn.
- Pump for about 5 minutes, stop, massage and rub the nipple then pump for 5 minutes. Your should pump for a total of 10 minutes on each breast.
- Longer pumping times does not produce more milk. Shorter more frequent pumping encourages milk production and better emptying of the breast.
- Drink a glass of water during your pumping session.

WARNING!

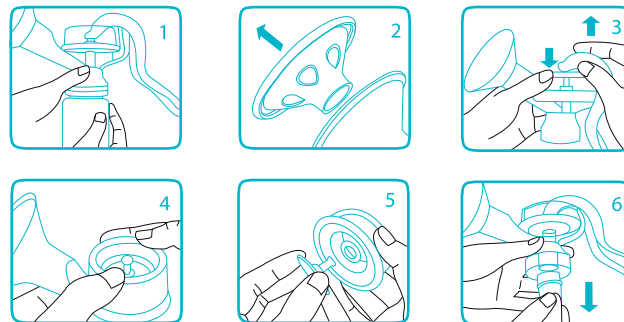
- Do not use aggressive detergents or disinfectants
- Do not wash the breast pump in a dishwasher.
- Keep the breast pump and its parts out of reach of the child.
- To be used only by an adult.
- Inspect the breast pump before use; In case of signs of deterioration: do not use it anymore.

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
CLEANING THE BREAST PUMP:

Before each use, all parts of the breast pump, including the bottle and its accessories, should be cleaned with soapy water, rinsed thoroughly and dried. Bottle and breast pump (remounted) can be sterilized in boiling water during 5 minutes or steam in a sterilizer.

DISMANTLING AND REASSEMBLING THE BREAST PUMP:

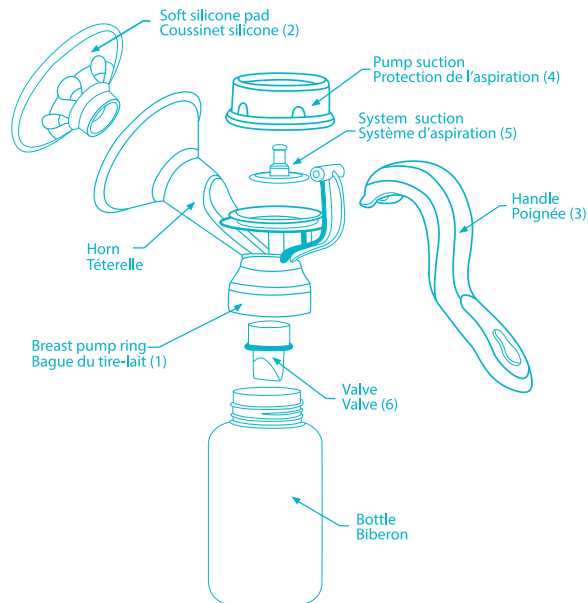- Unscrew the ring holding the breast pump on the bottle (1).
- Remove the silicone bubble pad (2) from the horn.
- Separate the handle from the assembly by pulling it upwards (3).
- Remove from the suction protection assembly (4) by pinching on both flexible sides.
- Remove the suction system (5) from its protection.
- Remove the valve (6) by pulling on it.
- After cleaning all the parts and drying, reassemble them in the opposite direction of disassembly.

S.E.B.I.R
54 rue Rouget de Lisle
F.93697 PANTIN CEDEX

www.dBb-remond.fr