---

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### ELECTRONICAL NASAL ASPIRATOR

---

FR

#### MODE D'EMPLOI

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Tél : +33(0)1 49 15 96 40

# MENU

Introduction	.....	01
Important information	.....	02
Product identification	.....	04
Batteries installation	.....	05
Instruction for use	.....	06
Care and maintenance	.....	08
Technical Specification	.....	09

# INTRODUCTION

## 1. Music Function

12 chord melody

## 2. Strong Power

Put aspiration tip into nostril, then you may use this device to remove nasal secretions continuously or intermittently.

## 3. Safe

The soft aspiration tip has a comfortable design to prevent the possibility for injury during operation.

## 4. Compact & Easy Design

This device is compact easy to hold and operate. It is also safe and comfortable for baby use.

## 5. Washable Head Module Design

Tips, collection cup & head part can be taken off from the device and washed. (Collection cup can also be disinfected by boiling water.)

## 6. Light Weight, Low Noise

Easy to use while the baby is sleeping.

## 7. Batteries

This device only needs to use two 1.5V AA batteries, it is convenient to change the batteries anytime, any where.

# IMPORTANT INFORMATION

The device is designed for using intermittent suction to remove nasal secretion and mucus in children (age 0~12 years old) at home environment.



**WARNING – Do not modify this equipment without authorization of the manufacturer**

1. Do not expose the device to extreme temperature, very high humidity level, or direct sunlight.
2. Avoid extreme stress, shock or accidental drop down.
3. Please replace all batteries when the power or/and the sound of the pump is getting weak while using the device.
4. It is recommended to remove the batteries if the device will not be used for a period of time.
5. The application of pressure by the device is intermittent. You shall keep pressing the power button to start the device to generate vacuum for removing nasal secretion and mucus. If you want to stop the suction, just need to release the power button to turn off the device.
6. The Nasal Aspirator is for cleaning/removing the nasal secretion only. Do not use this device other than the intended purposes.
7. During the use of the device, the baby should not hold his breath, close his mouth or swallow his saliva to avoid vacuum pressure cause eardrum unbalanced.
8. If the baby does not feel well when using the device, stop and consult a doctor immediately.

# IMPORTANT INFORMATION

9. Do not apply excessive force when placing the aspiration tip into the nasal cavity.
10. This device is for home use only.  
Please consult a doctor in case of nasal diseases. Clean the aspirator tip prior to each use.
11. The Intended Operators for this device shall be parents or adults who can follow the instructions and notices in user's manual to operate the device.  
The device is not allowed to be used by children themselves. Keep the nasal aspirator away from the children. Do not let them operate the device themselves.
12. DO NOT use the Nasal Aspirator when patients have the following contraindications:
  - Nasal bleeding
  - Acute head, facial, or neck injury
  - Coagulopathy or bleeding disorder
13. Before using the device, please read this User's Manual carefully. Wrong use of the device may cause one of the following hazards/complications:
  - Mechanical trauma
  - Nasal irritation/bleeding
  - Mucosal hemorrhage
  - Discomfort and pain
14. Please keep the replaceable tip at where children and pets cannot access to avoid risk of them been swallowed.
15. The tip may cause potential allergic reactions.

# PRODUCT IDENTIFICATION

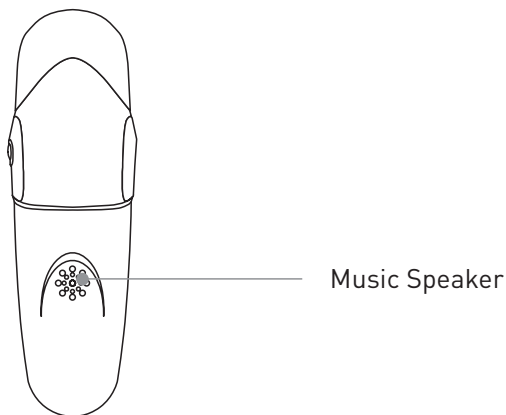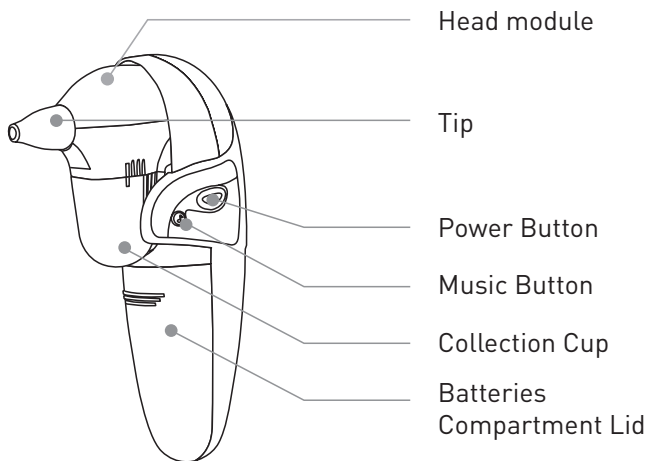
# BATTERIES INSTALLATION

## Replacing the batteries :

1. Gently slide the batteries cover back.
2. Carefully remove the old batteries and properly discard.
3. Insert or replace 2 x 1.5 V AA alkaline batteries into the batteries compartment, ensuring to match the indicated polarity symbols. Always use new batteries.
4. Replace the batteries cover.

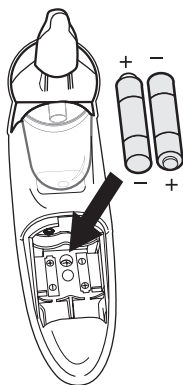## NOTE: Batteries-operated

1. Please properly dispose of the batteries away from small children and heat.
2. It is recommended to remove the batteries if the unit will not be used for an extended period of time.
3. For long durations of non-operation, please remove all batteries from the device.
4. Batteries must be disposed of in accordance with local environmental and institutional policies.
5. Dispose of used batteries in accordance with the applicable legal regulations. Never dispose of batteries in the normal household waste.

Please recycle

# INSTRUCTION FOR USE

1. Choose the correct tip to use, according to shape and size of the nostrils of your baby. The large tip is recommended for use when the mucus is in a more solid form. The narrow tip is recommended for use when the mucus is loose and in a liquid form.


**large tip**


**narrow tip**

2. Be sure to place the tip securely on the aspirator. Make sure tip is completely over the rib before using.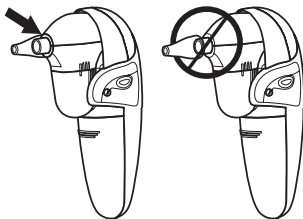


3. Press and hold the power button to allow the suction to clear the mucus.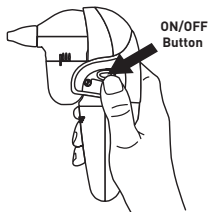


# INSTRUCTION FOR USE

4. Hold the baby upright and insert the tip into one nostril of the baby. Be careful not to place the tip too far into the nostril or press against the inside wall of the nostril. Be sure to keep the tip of the aspirator at a 90° angle. Do not point it up into the nostril. Allow the suction to clear the mucus. Repeat in other nostril and as needed.


The mucus may not immediately appear in the collection cup. After each use it is recommended to give a quick clean of the tip by removing the tip from the unit.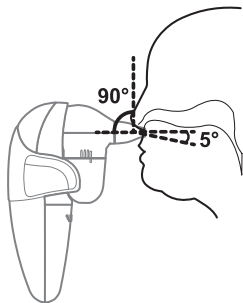

5. Release Power button to stop suction.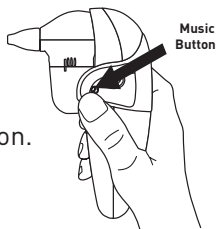

6. Our device provides the unique design of the chord ring and 12 worldwide children's ballads. Press the music button to start and press it to select the desired melody. After using the device, press the music button for 1 second to turn off the melody.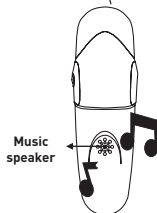

# CARE AND MAINTENANCE

1. Separate the tip and the head module from the base of the aspirator by gently pulling it apart.
2. Unscrew collection cup by turning it clockwise.




## Cleaning process of Collection cup, Head Module and Aspiration tip :

Washing the collection cup, head module and aspiration tip in water for 3 minutes. Then rinse these parts with running hot tap water for 3 minutes and air dry. After the cleaning process, these parts shall be disinfected by boiling water.

**DO NOT wash this device with any kinds of soap or cleaner. Please just wash this device with water only.**

### Unit Cleaning :

Wipe the unit with a clean soft dry cloth.

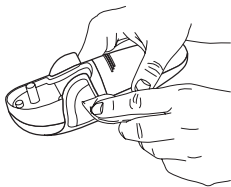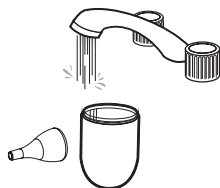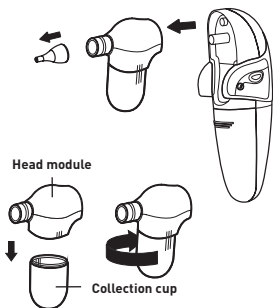# TECHNICAL SPECIFICATION

**Effective Vacuum Pressure** : 350mmHg (60kPa)

**Sound Pressure Level** : 75~80 dB

**Aspiration Tip** : Soft Silicone

**Motor** : 3V/DC

**Operating environment** : 16°C~35°C (60.8°F~95°F)

with relative humidity up to 85% (non condensing)

**Storage/ Transportation environment** : -10°C~+55°C

(-14°F~+131°F) -10°C~+55°C (-14°F~+131°F)

**Power Source** : 2 x 1.5 V AA alkaline batteries

**Weight** : approx. 146g (without batteries)

**Dimensions** : approx. L 98.5 mm × W 45.3 mm × H 143mm

**Batteries Operation Time** :

About 90 minutes (Depends on different brand of batteries.)

**Typical Service Life** : 3 years

**Guarantee** : 2 years

To dispose of the product responsibly :

- Do not throw it away with your household waste.
- For the correct collection and treatment of this product take them to the collection points designated by your local authority.

Model : NS2

Electronic nasal aspirator

Distributed by : S.E.B.I.R Remond

54 rue Rouget de Lisle

93697 PANTIN Cedex, France

Phone : +33(0)1 49 15 96 40

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54 Rue Rouget de Lisle  
93697 PANTIN CEDEX

+33 (0)1 49 15 96 40

[www.dBb-remond.fr](http://www.dBb-remond.fr)

[contact@dbb-remond.fr](mailto:contact@dbb-remond.fr)

[www.facebook.com/dBb.Remond](https://www.facebook.com/dBb.Remond)

[www.instagram.com/dbbremond](https://www.instagram.com/dbbremond)