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SUMMARY

CONGRATULATIONS	P. 46
SAFETY INSTRUCTIONS	P. 47-48
BREAST PUMP & ACCESSORIES	
Included elements	P.49
Pumping kit	P.50
ASSEMBLY OF THE PUMPING KIT	
How to install the valve with membrane in the pumping head?	P.51
How to install the silicone cushion?	P.52
How to install the diaphragm?	P.53
How to assemble the pumping head to the pipe?	P.54
CLEANING INSTRUCTIONS	P.55-58
ABOUT THE DBB BREAST PUMP	
dBb breast pump engine block	P.59
Symbols of the LCD screen	P.59
Use of the breast pump	P.60
The pumping modes	P.61
Standby mode	P.62
Memory	P.62
Function M (My favorite mode)	P.63
How to set the clock?	P.64
How to set the alarm?	P.65
How to mute sounds?	P.66
POWER OPTIONS	P.67-68
USE OF THE MANUAL BREAST PUMP	
How to assemble the manual breast pump?	P.69-70
Running of the manual breast pump	P.71
PRESERVATION OF THE BREAST MILK	
How to store breast milk?	P.72
How to freeze breast milk?	P.73
How to heat breast milk?	P.73
TROUBLESHOOTING	P.74-76
EMC/TECHNICAL DESCRIPTION	P.77-81
WARRANTY	P.82
TECHNICAL INFORMATIONS	P.83
SYMBOLS LEGEND	P.84-85

CONGRATULATIONS

Congratulations on the purchase of your new dBb Remond electronic dual suction breast pump.

The breast pump is intended to be used by nursing women on many occasions:

- to collect breast milk when you are far from your baby, for example, when you are at work or if the father wants to feed the baby.
- to help relieve the symptoms of engorgement by collecting milk. For example : when your baby diversifies his diet.
- to facilitate the healing process, in the event of mastitis, by the aspiration of breast milk from inflamed breast.
- to relieve flat or inverted nipples.
- to maintain your milk supply if you need to temporarily stop breastfeeding. For example: collecting inappropriate milk during medical treatment.
- to allow mothers to give breast milk to babies who cannot breastfeed for a variety of reasons, such as breastfeeding problems, cleft palate, or prematurity.

The dBb Remond double suction electronic breast pump offers many practical features for moms:

- 3 suction modes close to the baby's suction:
 - > pumping 1 - massaging and stimulating;
 - > pumping 2 - fast but gentle;
 - > pumping 3 - natural and regular;
- milk return protection.
- silent motor for discretion.
- compact and robust.
- 2 power options to meet your needs:
 - > AC / DC adapter (included);
 - > Built-in rechargeable Li-ion battery;
- conversion to manual breast pump in seconds (manual handle included).

Indication of use:

The dBb Remond double suction electronic breast pump is an electrical device for personal use. It helps extract milk from the breasts of lactating women. This device is not intended for hospital use.

SAFETY INSTRUCTIONS

The dBb Remond electronic dual suction breast pump is designed so that breastfeeding women can extract and collect breast milk. It is intended for a single user and should never be shared or resold, this would pose health risks and would void the warranty.

READ INSTRUCTIONS BEFORE USE

DANGER !

To reduce the risk of electric shock:

- always unplug the breast pump immediately after use it.
- do not use it while bathing or showering.
- do not place or store the breast pump in a place where it could fall in a bathtub or sink.
- keep the device dry. Never immerse it in water or other liquids.
- do not reach for an electrical product that has fallen into water. Unplug immediately.

WARNING !

- never leave a connected appliance unattended.
- keep the pump out of reach of babies, children or pets. Small parts and components may pose a swallowing, choking or strangulation hazard.
- do not use this breast pump if the cord or plug is damaged, if it does not work correctly, if it has been dropped or damaged, or if it has fallen into water.
- keep the cord away from hot surfaces.
- do not use while you sleep or if you are drowsy.
- do not drop or insert any other object into openings or pipes.
- check that the voltage of the AC / DC adapter is compatible with the power source.
- plug the AC / DC adapter into the power outlet before plugging in the cord on your electronic breast pump.
- dBb Remond supplied an AC / DC adapter with this device. The use of any other AC / DC adapter not approved by the manufacturer will void the warranty.
- if you are infected with hepatitis A or hepatitis B or with HIV (virus immunodeficiency), the pumping of breast milk by the electronic dual suction breast pump does not reduce or eliminate the risk of the virus being passed to your baby through breast milk.
- wash the parts of the breast pump before each use.
- do not drive a vehicle when using the electronic double suction breast pump during hands-free suction.
- do not try to modify the device. All repairs must be carried out exclusively by an authorized technical service.
- use of this equipment adjacent to or stacked with other equipments should be avoided because it could result in improper operation.

SAFETY INSTRUCTIONS

- anti-backflow is the essential performance of the dBb breast pump. If any moisture or milk is found in the pipes while using or after using the breast pump, stop using it.
- do not use antibacterial or abrasive cleaner/detergent when cleaning breast pump or breast pump parts.
- Never put the pump unit into water or sterilizer as it can cause permanent damage to the pump.
- using a breast pump on an aircraft is not recommended. The cabin pressure may affect the breast pumps performance.
- If there is any potential allergic reaction, stop using our products and find a medical advice immediatly.

CAUTION:


- do not use this product for any other purpose than as indicated in this manual.
- make sure all of the parts are assembled correctly before use.
- stop using the breast pump immediately if your breasts or nipples are painful or if you feel discomfort. In case of problems, contact a professional help.
- please wash your hands before using this breast pump.
- do not drop or strike the device. It could cause damage not covered by warranty.
- do not touch the product with bare hands immediately after sterilization by boiling water to avoid scalds and burns.
- do not overheat materials of the breast pump; to prevent premature aging.
- do not use any accessory, part or other that are not recommended by dBb Remond. That will void the warranty.
- do not connect the pump or any of its accessories to any equipment not recommended by dBb Remond.
- do not store the product in a dusty, or damp environment, or in direct sunlight.

Note: When disposing of the breast pump or its accessories, please consult your local authorities for the proper disposal method for potentially biohazardous parts and accessories, or if there are other regulations.


SAVE THESE INSTRUCTIONS
along with your receipt for future reference.

BREAST PUMP & ACCESSORIES

INCLUDED ELEMENTS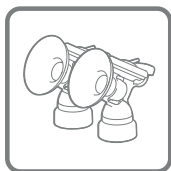


**2 PUMPING
KITS**

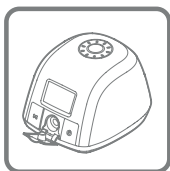
ENGINE BLOCK


**2 L.O BOTTLES
4 OZ**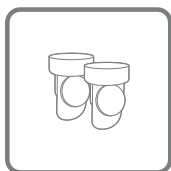


4 VALVES

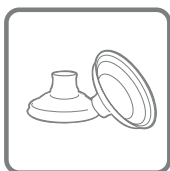2 SILICONE CUSHIONS

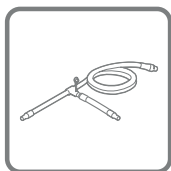
PIPES

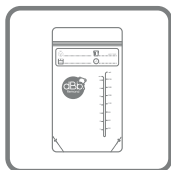**6 BAGS TO
PRESERVE MILK**


**MANUAL
HANDLE**

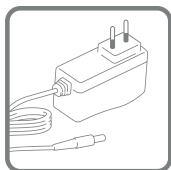9V AC/DC OUTLET


DRYING NET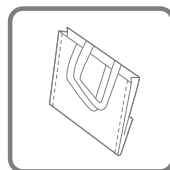

CARRYING BAG

BREAST PUMP & ACCESSORIES

PUMPING KIT

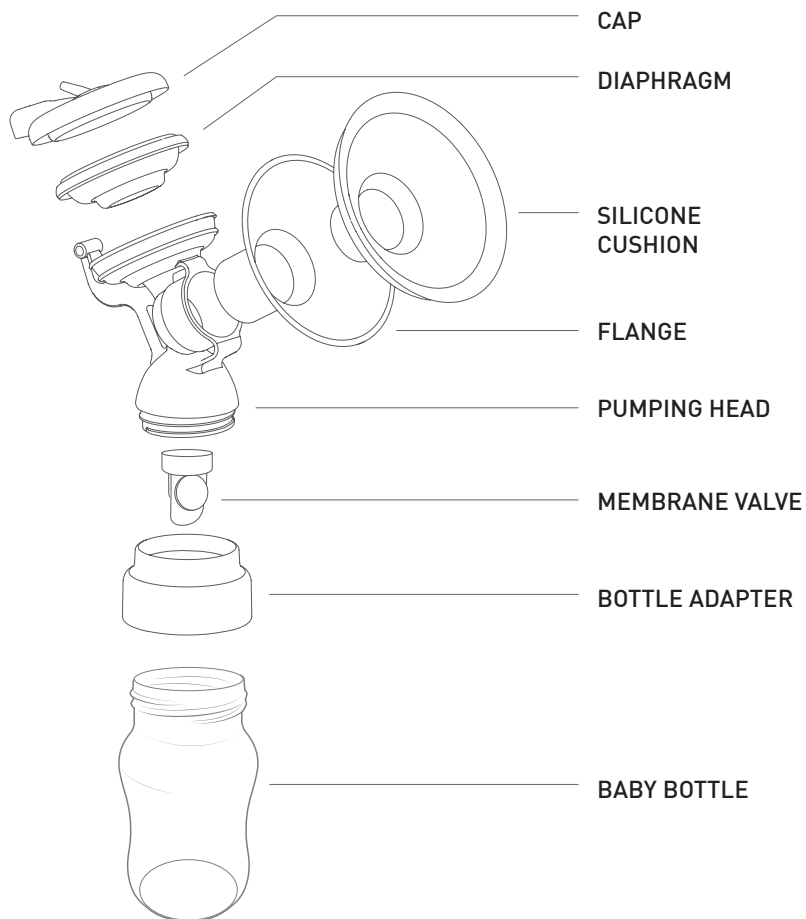ASSEMBLY OF THE PUMPING KIT

HOW TO INSTALL THE VALVE WITH MEMBRANE IN THE PUMPING HEAD?

Wash your hands thoroughly before handling.

Hold the pumping head with one hand and make sure that the part with the screw thread is facing up.


Insert the valve in the lower part of the pumping head.

Check that the valve diaphragm is flat. A rolled up membrane will cause loss of suction.


Check that the valve is correctly installed on the pumping head.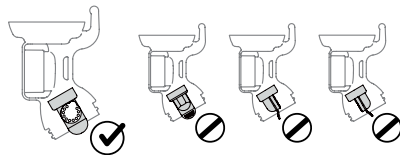


Make sure that the valve is inserted exactly as shown in order to maintain pumping efficiency.

HOW TO INSTALL THE MEMBRANE TO THE VALVE?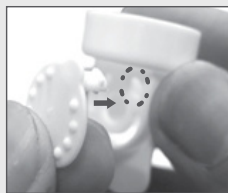

Insert the small rod of the silicone membrane into the opening at the top right of the valve.

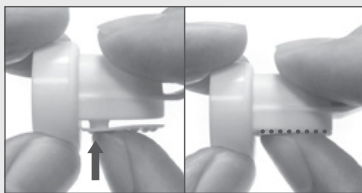
Press firmly on the diaphragm to make sure the rod is fully inserted and snug against the valve.

The assembly is finished.

ASSEMBLY OF THE PUMPING KIT

HOW TO INSTALL THE SILICONE CUSHION?

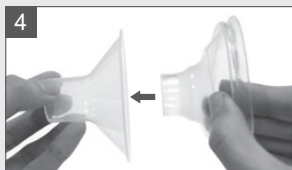If desired, place the silicone cushion in the flange.

Note: Your electric breast pump will work with or without the silicone cushion. Choose according to your comfort.


Push the center of the silicone cushion into the flange.

Attach the edge of the cushion to the edges of the flange.

Make sure that the edge of the cushion is in place.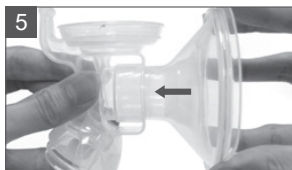

Insert the flange into the pumping head.

ASSEMBLY OF THE PUMPING KIT

HOW TO INSTALL THE DIAPHRAGM?

Insert the diaphragm into the location above the pumping head.


Note: Do not reverse the order of steps 6 to 9. This will cause loss of suction when pumping.

Press the edges of the diaphragm for a good fit.

Make sure the diaphragm is securely positioned. This is crucial for obtaining adequate suction and preventing backflow of the milk.

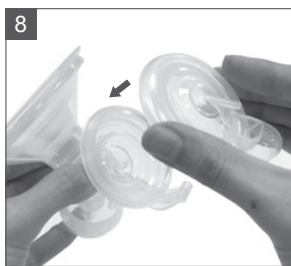Replace the cover on the diaphragm.


Once the cover is installed and aligned, press firmly on it so that it adheres well.

The pumping head is assembled.


ASSEMBLY OF THE PUMPING KIT

HOW TO ASSEMBLE THE PUMPING HEAD TO THE PIPE?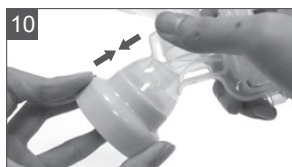


Screw the screw ring on the pumping head.

Screw the assembly on the body of the baby bottle.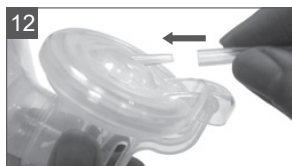

Put the hose on the end cap of the cover.


Press firmly on the assembly to guarantee watertightness.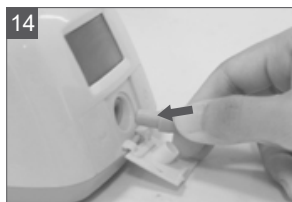


Note: Make sure there is no moisture in the hose.

Make sure that the end of the pipe is correctly inserted into the hole of the dBb breast pump's engine block.

USE IN SINGLE PUMPING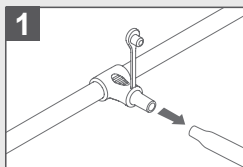

Remove one of the pipes from the separator by pulling it.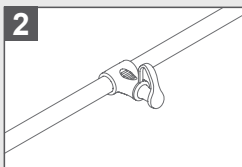

Close the separator cover to ensure watertightness.

The breast pump is ready for single pumping use.

Note: Using the electronic breast pump after converting from single to double or double to single pumping may result in a temporary drop in suction or over-suction, respectively. Please exercise caution and adjust the pump to your comfort and efficiency level.

CLEANING INSTRUCTIONS

Disinfect before first use:

Before using your dBb Remond double electronic breast pump for the first time, disassemble and boil only the parts of the pumping kit in water for 10 minutes.

- make sure to use enough water to keep all parts immersed when boiling.
- do not boil the silicone pipe or pipe separator.
- remove all sterilized parts using pliers. To avoid burns, do not touch the product with bare hands immediately after sterilization.
- dry the parts completely in the open air or with a clean cloth before putting them on to assemble. Wet components can affect the performance of your electronic breast pump. Check that all parts are completely dry before use.

Before each use:

- wash your hands thoroughly with soap and warm water before handling the pumping kit, the double electronic breast pump, or any part or accessory.
- make sure all parts are clean and dry. Assemble your pumping kit. Inspect each component for any cracks, tears, discoloration, or deterioration. Please discontinue use and replace the part immediately if damage is found. Inspect whether the kit or pipes has become moldy or soiled during storage. If your pipe is moldy, discard and replace immediately.
- clean if using a share pump. Wipe pump dials, Power Button, and pump housing with a low-level disinfectant such as Ethyl or isopropyl alcohol (70 – 90 %) is recommended.

Special cleaning instructions for the valve:


The membrane is specially designed with a knot preventing it from detaching.

To clean the valve, pull the membrane slightly until it stops.

Clean the valve in hot soapy water, then rinse thoroughly with clean and lukewarm water. After cleaning, fully press the end of the membrane to replace it correctly and make sure it rests flat against the valve opening.

Caution: Be careful when cleaning the valve. If damaged, your breast pump will not function properly.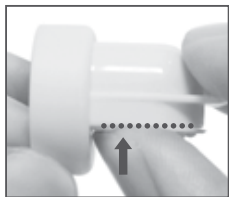

CLEANING INSTRUCTIONS

After each use:

1. keep your milk safe by closing the bottle or sealing the bag. Label it with the date and time, and immediately place in a refrigerator, freezer or cooler bag with ice packs.
 - if your milk will be stored in a hospital or daycare, add a name to the label.
2. clean the parts of the breast pump. Especially if you are using a shared pump, it is recommended to wipe the dial, the keys and the body of the breast pump with a low level disinfectant such as ethyl or isopropyl alcohol (70 to 90%).
3. disassemble and inspect the pumping kits. Disassemble the breast pump pipes and leave them aside. Separate all parts that come into contact with breast / breast milk (for example, silicone cushions, valves, membrane, etc.).
4. rinse the elements of the pumping kit under running water to remove the rest of the milk.
5. as soon as possible after pumping, clean the pumping heads that come into contact with the breast / breast milk in one of the following ways:

Hand cleaning:

- use a sink. Place the various items in a clean basin used only for washing infant feeding equipment. Do not place pump parts directly in the bowl, as germs in sinks or pipes could contaminate the breast pump.
- add soap and water. Fill the sink with hot water and add soap.
- scrub the items with a clean brush used only to clean your child's items.
- rinse by keeping items under running water or by immersing them in water in a basin used only to clean your child's items.
- allow to air dry well. Place the parts of the breast pump, the bowl and the brush on a clean, unused cloth or paper towel in a place protected from dirt and dust. Do not use a cloth to rub or wipe the elements, as this could contaminate them. dBb also provides a drying bag for your convenience to air dry all of the different parts of the breast pump.

Clean in a dishwasher:

- place the disassembled parts of the breast pump in the dishwasher. Be sure to place small items in a closed basket so that they do not end up in the dishwasher filter. If possible, run the dishwasher with hot water and a heated drying cycle (or disinfection setting); it can help kill more germs.
- wash your hands with soap and water before removing and storing the cleaned items. If they are not completely dry, place them on a clean, unused tea towel or paper towel to air dry them before storing or use the drying bag provided. Do not use a cloth to rub or wipe items, as this may facilitate contamination of the items.

CLEANING INSTRUCTIONS

6. if you are using a bowl or brush to clean the parts of the breast pump, rinse thoroughly and allow to air dry after each use. Consider washing them daily, either in a dishwasher with hot water and a heated drying cycle, if they are dishwasher safe, or by hand with soap and lukewarm water.
7. wipe the outside of the hose with a clean, damp cloth. Avoid running water or other liquids inside the pipe.
 - If liquid is inside the pipe, rinse it with hot water.Then hang it in the open air for complete drying before use. Do not insert any object into the pipe.

For additional protection, disinfect!

To further eliminate germs, disinfect the parts of the breast pump at least once a day. Disinfection is especially important if your baby is less than 3 months old, born prematurely, or has a weakened immune system due to illness or medical treatment (such as chemotherapy for cancer).

Daily disinfection is not mandatory for older, healthy babies if the items are cleaned thoroughly after each use.

Disinfect all the elements (even the brush and the bowl!)

Using one of the following options:

Note: If you are using a dishwasher with hot water and a heated drying cycle (or disinfection setting) to clean your child's items, the separate disinfection step is not necessary.

1. clean up first. The parts of the breast pump, the brush and the bowl should not be disinfected until they have been cleaned.
2. disinfect the breast pump, brush and basin using one of the following options.
 - a) steam:
 - use a microwave oven or sterilizer according to the manufacturer's instructions.
 - b) boiling:
 - place the disassembled items in a saucepan and cover with water. Put the pan on fire and bring to the boil for 10 minutes. Remove the elements with clean pliers to avoid the risk of burns.
3. allow to air dry completely. Place the parts of the disinfected breast pump, the bowl and the brush on a clean, unused cloth or paper towel in a place protected from dirt and dust. Do not use a cloth to rub or wipe the elements, as this could transfer germs. You can also use the drying bag provided to air-dry the parts of the breast pump.

CLEANING INSTRUCTIONS

Storing your breast pump:

Let the parts of the breast pump, the brush and the bowl dry completely in the air before putting them away to avoid the proliferation of germs and molds. Once completely dry, the items must be stored in a clean and protected place to avoid contamination during storage.

- wash your hands with soap and water.
- assemble the cleaned and dried parts of the breast pump.
- store the reassembled breast pump in a clean and protected place as inside from a bag or its original box. Also store the bowl and brush in a clean place.


Pipe cleaning instructions:

- the breast pump pipes are designed to not touch the pumped milk and do not need to be cleaned regularly.
- if your pipe contains water droplets at the end of a pumping session, remove it and inspect it thoroughly maybe the diaphragm needs to be replaced. Rinse the pipe with clean hot water. Let it dry completely before to use it again.
- if your pipe contains mold, discard it immediately as it is difficult to clean properly. Replace it with a new pipe and check if the diaphragm must be replaced.
- if the outside of your pipe is dirty, wipe it with a low-level disinfectant such as ethyl or isopropyl alcohol (70 - 90%) is recommended.
- do not insert any object into the tube.

ABOUT THE DBB BREAST PUMP

DBB BREAST PUMP ENGINE BLOCK

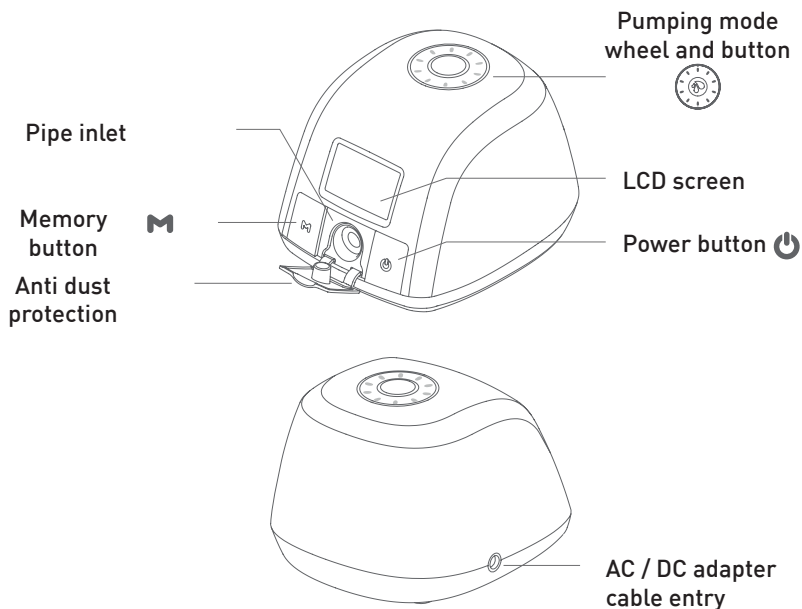SYMBOLS OF THE LCD SCREEN

ABOUT THE DBB BREAST PUMP


USE OF THE BREAST PUMP

Hold the breast pump on your breast so that your nipple is centered into the flange.

1. Open the dust cover and insert the pipe into its inlet.

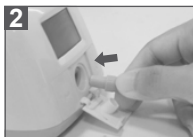2. Make sure the breast pump pipe is properly installed.

Press the power button  to start pumping milk.

The LCD screen will display the time of your last pumping session for a few seconds. The breast pump will automatically start in pumping mode 1 for 15 seconds, then in pumping mode 2 .

ABOUT THE DBB BREAST PUMP

THE PUMPING MODES

There are several intensity levels ranging from 1 to 8 for pumping mode 2, and from 1 to 10 for pumping mode 3.

Adjust the intensity to your comfort level during use by turning the scroll wheel  clockwise to increase the intensity, or counterclockwise to decrease the intensity.

Depending on your sensitivity, you can choose pumping mode 2  or pumping mode 3 . To switch between them, press the suction mode button once .

To check the pumping mode you are in, the pumping mode icon appears on the LCD screen. The drop icon indicates that you are in suction mode 2 (fast and gentle) .

The double drop icon indicates that you are in suction mode 3 (natural and regular) .

There are 8 intensity levels for pumping 2, and 10 intensity levels for pumping 3.

Turn off the breast pump by pressing the button .

ABOUT THE DBB BREAST PUMP

STANDBY MODE

Once the breast pump is turned off, the LCD screen will stay on for a few seconds. The display will show the total pumping time, then automatically switch to standby / clock mode or alarm mode if it is activated.

After disconnecting the pipe from the engine block, make sure to cover the pumping socket with the dust cover.

Note :

- *make sure there is no more suction on the nipples before removing the breast shield. If necessary, stop the suction by pressing on the breast near the flange so that the air enters between the two in order to detach the breast shield from your breast.*
- *if the electronic breast pump is on, but unattended for more than 30 minutes, it will stop automatically.*

MEMORY

Each time you select a pumping level (in any pumping mode), your breast pump stores it until you change it. This feature becomes useful once you have become familiar with your breast pump. The breast pump memorizes the level chosen during the previous session. Very useful when you have found the one that suits you best.

ABOUT THE DBB BREAST PUMP

FUNCTION M (MY FAVORITE MODE)

Memorize your aspiration mode by pressing the **M** button. It is extremely useful if you usually use pumping mode 3 to collect your milk. If you only use pumping mode 2, you don't need to use the **M** button.

Set my preferred mode:

1. after a few pumping times, you will notice that the pause time is about nearly the same at each suction session.
2. start collecting your milk as usual. Pumping mode 1 starts for 15 seconds, then automatically switches to pumping mode 2 and finally to level of suction you used last.
3. press the pumping mode button  when your milk starts to drop to switch to suction mode 3.
Adjust the desired suction level with the scroll wheel.
4. press and hold the **M** button for three seconds, until the M icon is flashing,  your preferred mode is now set.

5. continue the session as needed.

To return to the preferred mode M:


1. turn on the breast pump .
2. press the **M** button (DO NOT hold for more than 3 seconds).
3. your breast pump automatically switches to the registered pumping mode.

Note: You cannot adjust the pumping level or switch between pumping 2 and 3 when you are in My preferred mode. If you want to change the pumping level, press the **M** button once to exit My Favorite Mode.

ABOUT THE DBB BREAST PUMP

HOW TO SET THE CLOCK?

1. in Clock / Standby mode, keep the **M** button pressed for 3 seconds to set the clock. A beep will sound and the AM / PM icon will flash.
2. to choose between AM / PM, turn the scroll wheel  clockwise or counterclockwise.
3. press the scroll wheel  once to validate the time display [AM / PM]. Time will flash.
4. to change the time, turn the scroll wheel  clockwise to increase (+) or counterclockwise to decrease (-) time.
5. press the scroll wheel  again to validate the hour and modify the minutes. The minutes will flash.
6. to change the minutes, turn the scroll wheel  clockwise to increase (+) or counterclockwise to decrease (-) the minutes.
7. press the scroll wheel  one last time to save.

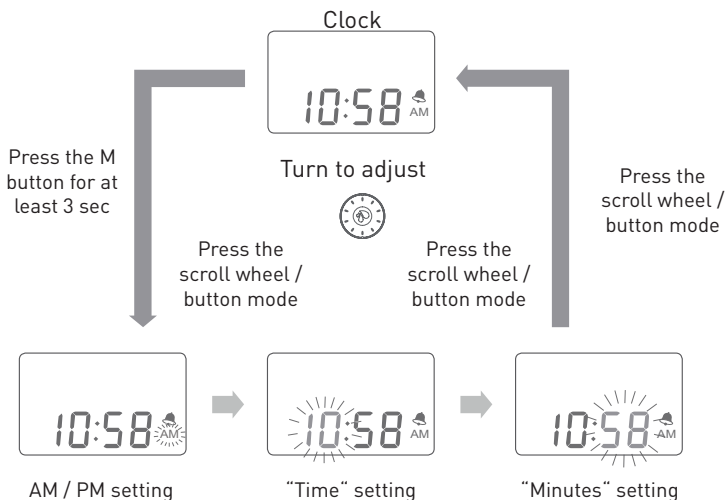ABOUT THE DBB BREAST PUMP

HOW TO SET THE ALARM?

The alarm mode is designed to remind you of your next pumping session. By keeping a regular pumping pattern, your body produces a good amount of milk. Once you have set the time interval between each pumping session, the countdown will automatically start at the end of the previous pumping cycle. An alarm will sound after the preset time has elapsed.


1. in Clock / Standby mode, press the **M** button to get access to the alarm settings.
2. the LCD screen will alternately display two series of duration: the interval duration of the alarm (which is the duration between each pumping session) and the duration of the next alarm (the time of your next pumping session). For example, if the time of power off of your last session is 10:58 and you want to perform next pumping session in 2 hours and 30 minutes, set the time alarm interval of 2 hours and 30 minutes. The next alarm time will be fixed at 1:28 p.m.
3. turn the scroll wheel  clockwise (+) or counterclockwise (-) to set the duration of the alarm interval. The setting increases (+) or decreases (-) in 15 minute intervals. Next time alarm will be automatically adjusted according to the last use of the milk tire.

Note: *The alarm interval time is the period from the end of the last session to the next deactivation time. It is not the time between the start of one suction session and the start of the next.*

4. press the scroll wheel  to go to the ON / OFF alarm parameter.
5. turn the scroll wheel  clockwise (+) or counterclockwise (-) to activate [ON] or deactivate [OFF] the alarm.
6. Press the scroll wheel  again to save your setting.

Note: *If no setting is made within 10 seconds in the alarm setting mode, the setting is saved automatically and the LCD screen returns to Clock / Standby mode.*

ABOUT THE DBB BREAST PUMP

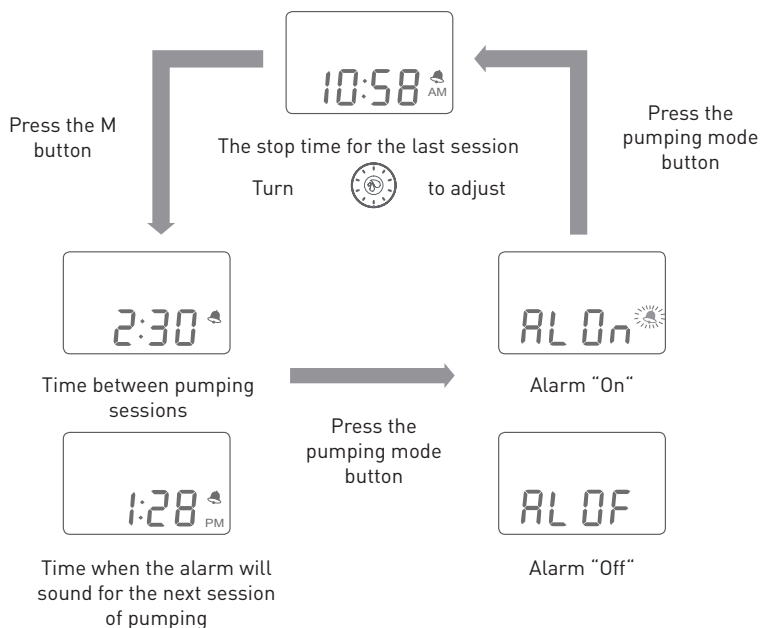HOW TO MUTE SOUNDS ?

The breast pump ringtones can be disabled in "alarm setting mode". "Alarm disabled" means the ringtones are disabled. "Alarm On" means the buzzer will sound.

Note: The alarm icon displayed on the screen means that the pumping tones will sound.

POWER OPTIONS

Your electric breast pump can be used with an AC or battery power mode.

AC/DC ADAPTER

Connect the AC / DC adapter to an electrical outlet.

Attach the power cord to the breast pump.

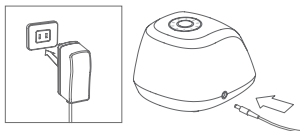
Caution: Electrical safety can only be obtained by using the AC / DC adapter supplied with the breast pump. The use of another adapter poses a safety risk and / or voids the warranty.

Danger: If the power adapter outer casings become loose or are separating, contact the device manufacturer immediately. Separation of the casing can result in electric shock.

LI-ION BATTERY

Your breast pump contains a rechargeable Li-ion battery.

- fully charge the battery for 12 hours before first use.
- you can use the breast pump with the power cord plugged into an external power supply. The battery will charge as soon as the engine block is connected, but not during its use.
- after the initial charge, each subsequent charge takes approximately 6 hours.
- for best results, keep your battery fully charged and recharge it before it drains completely.
- keep the breast pump in a cool place.
- When the «low battery» light  appears, recharge the battery as soon as possible by plugging the ac / dc adapter into the wall outlet and the other end of the power cord into the breast pump.
- your breast pump may not work immediately on battery if it has been stored in a heated place. If this is the case, you can let the battery cool while using it on an external power supply.

Note :

- *the rechargeable Li-ion battery used in this device may present a risk of fire or chemical burn if improperly treated.*
- *do not disassemble the battery.*
- *do not place the battery near a heat source or fire.*
- *dispose of the used battery correctly.*
- *do not throw the battery into fire.*
- *keep away from children.*
- *do not try to open the breast pump to replace the battery or for any other reason. This will void the warranty.*

Battery charge status codes :

External power source connected.

Battery charged.

(flashing)

Low battery, needs charging.

(flashing)

Battery drained.

Fully charge the battery for 6 hours.

Operating time remaining (approx.) :

Full bar : 2 hours.

2 bars : 1,5 hours.

1 bar : 1 hour.

(flashing)


1 bar flashing : 30 minutes.


- *Keep the battery charged at 50% capacity and at room temperature (20-25 ° C / 68-77 ° F) for long-term storage.*
- *We recommend charging the battery every 3 months to avoid excessive discharge.*
- *Battery performance will deteriorate after prolonged use or storage.*
- *Battery life : Continous charge and discharge for 500 cycles, the capacity will be \geq 70% of initial capacity.*

USE OF THE MANUAL BREAST PUMP



HOW TO ASSEMBLE THE MANUAL BREAST PUMP ?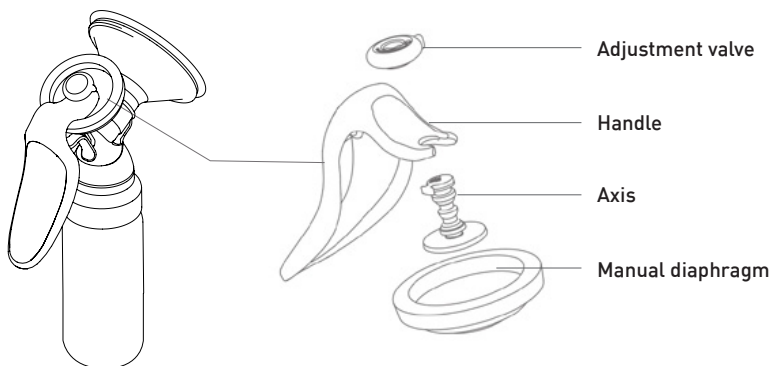

For the installation of the valve with membrane and the silicone cushion, please refer to the section "Assembly of the pumping kit".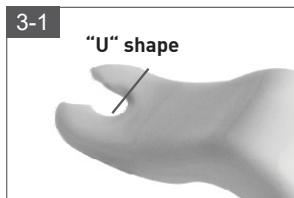

USE OF THE MANUAL BREAST PUMP

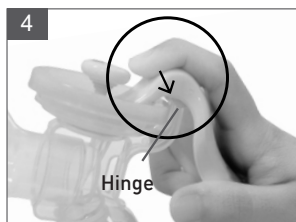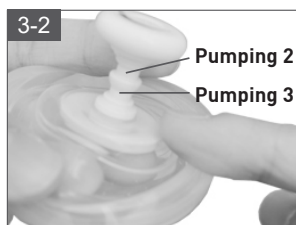There are 2 notches on the axis of the control valve. The upper notch corresponds to pumping mode 2 (fast) and the lower notch reproduces pumping mode 3 (natural).

Press the handle on the hinge of the pumping head until it snaps into place.

Note: There are two notches for attaching the manual handle. You can switch between these two settings by moving the axis of the “U» shaped tip away from the handle to dislocate the connection. Reattach the tip of the handle using a different notch on the axis of the control valve.

Pumping 2:

This setting is for quick and not prolonged pumping. Often when a baby is hungry or starts feeding, the pumping is fast and not intense to stimulate the breast milk flow.

Pumping 3:

This setting is for longer and more intense pumping. When the mother's milk begins to flow into the baby's mouth, the baby adopts a more regular and natural suction rhythm.


Note: You can use the manual handle in one of the two pumping modes. For best results, use a mode or combination of pumping modes that best matches your baby's suction and swallowing habits.

USE OF THE MANUAL BREAST PUMP


RUNNING OF THE MANUAL BREAST PUMP

Hold the manual breast pump against your breast so that the nipple is centered inside, then press the handle toward the bottle to test for pumping.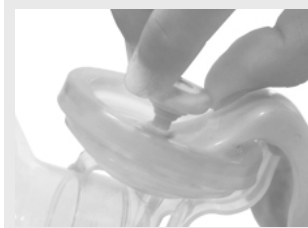

Note: If adjustment is necessary, turn the round knob on the adjustment valve to a comfortable pumping level.

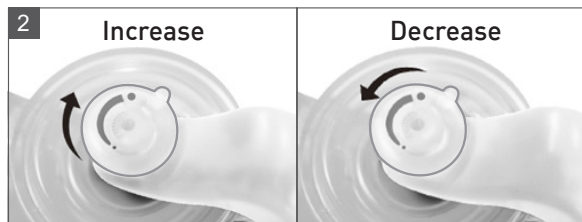You can adjust the pumping level by turning the round valve located at the top of the shaft. Turn clockwise to increase the pumping and counterclockwise to decrease it.

Note: Do not rotate the control valve too much as this will cause the valve to bulge and air leakage, which would create a suction problem. Make sure the adjustment valve is properly aligned on top of the shaft so that it turns smoothly. If necessary, readjust the control valve.

PRESERVATION OF BREAST MILK

HOW TO STORE BREAST MILK?

The bags provided with the dBb electronic breast pump are the most convenient and hygienic way to store, transport and warm breast milk. There are 6 bags but they are also sold separately by lots of 25 units.

Good hygiene is essential to guarantee the safety of preserved breast milk and to be able to give it to your baby. Wash your hands thoroughly with soap and water before touching the storage bag. Dry your hands with a perfectly clean towel or paper towel. Avoid touching the inside of the storage bag. From a hygienic point of view, the dBb breast milk bags are designed for single use. Never reuse the same bag as this may cause cross-contamination.

Storage of freshly collected breast milk (For healthy term babies)			
Ambient temperature 16-25°C.	refrigerator 4 ° C or colder.	Freezer -18 ° C.	Thawed breast milk in the refrigerator.
Suitable for consumption during 4 hours.	Suitable for consumption during 2 days.	Suitable for consumption during 4 months.	Suitable for consumption during 24H.
* Up to 6 hours for milk extracted under very clean conditions.	* Up to 5 days for milk extracted under very clean conditions.	* Up to 9 months for milk extracted under very clean conditions.	At room temperature: for 2 hours. Do not refreeze!

* Instructions for collecting milk under very clean conditions:

Before using the breast pump, mothers should wash their hands with soap and water or an alcohol-based hand sanitizer. The parts of the breast pump, the bottles and your environment must be clean.

These rules for storing and defrosting breast milk are recommendations. Contact your lactation consultant or breastfeeding specialist for more information. Keep breast milk in the coldest part of the refrigerator (above the vegetable drawer).

Do not add freshly collected milk to milk already in storage.

PRESERVATION OF BREAST MILK

HOW TO FREEZE BREAST MILK ?

Freeze breast milk in milk storage bags. Do not fill the bag beyond the maximum graduation as breast milk takes volume when frozen. Write the date of extraction and the quantity of milk on the bag.

Do not use breast milk if the parts are damaged.

HOW TO HEAT BREAST MILK ?

Do not defrost or warm breast milk in the microwave or in a pot of boiling water to avoid loss of vitamins, minerals and other essential components, and the risk of burns. Defrost your breast milk storage bag in the refrigerator (approx. 8 hours).

To warm to a body temperature (approx. 37 ° C) relative to the breast, place the storage bag in a bowl of hot water, in a bottle warmer or under a hot water tap. Then pour the contents into baby bottle and check the temperature of the breast milk before giving food to your baby.

The storage bags are made of recyclable Polyethylene.

About the storage bags:

For your child's safety and health

WARNING!

- this is a single use product and shall be disposed of after first use.
- product intended to be used without the need to clean before first use.
- for use with Breast Milk only. Not to be used for mixing formula milk.

TROUBLESHOOTING

ELECTRIC SERVICE INTERRUPTION

If you experience an interruption in the power supply due to a power surge or other factors, during the normal operation of your dBb double electronic breast pump, try the following solutions:

- disconnect the AC / DC adapter from the electrical outlet and wait 5 seconds.
- reconnect the AC / DC adapter to the mains socket.
- if the problem persists, try another electrical outlet.

LOWER PUMPING

If the breast pump has lost suction, the problem may not be the engine block:

- check that the breast pump is correctly assembled.
- center your nipple in the breast shield and make sure the area is perfectly waterproof between the breast shield and your skin.
- press the diaphragm firmly into the cup and make sure there is no gap.
- check the valve and make sure it is positioned correctly.
- inspect the membrane and valve very carefully to detect any tears or deformation. This component is essential for obtaining adequate suction; even the slightest tear will have a negative impact on the performance of the breast pump. If it is damaged, replace the valve with one of the replacement valves supplied with your breast pump.
- If problems occur on one side during pumping, swap the pipes to see if the problem occurs on the other side.
If so, the problem may be with the pump. Contact technical support for assistance.
If not, follow the steps below to resolve the problem.

If the two pumping heads of the breast pump have a loss of suction:

- detach the pipe from the pumping head cover and place your finger on the mouth of the pipe.
- if you feel a suction, the electric pump works correctly.
- if you do NOT feel suction or it has decreased, contact the dBb Remond customer service.
- check that the pipes are not torn or blocked.

NO RESPONSE ON THE USE OF THE WHEEL / SUCTION MODE BUTTON

You CAN NOT set the suction level or switch between gentle and natural suction while in "My Favorite Mode". If you have to do it, stop "My Favorite Mode" by pressing the M button.

TROUBLESHOOTING

THE PUMP DOESN'T WORK DESPITE THE TURNED DEVICE

If the power symbol  is flashing or if you see "Err1" and hear 4 beeps (under Alarm ON), contact the dealer where the breast pump was purchased.

PRESENCE OF BREAST MILK IN THE PUMPING HEAD

If breast milk does not seem to flow quickly enough in the bottle, reduce the pumping setting of your electronic breast pump to allow the milk to flow before gradually increasing the pumping to the initial setting.

WHERE TO PURCHASE REPLACEMENTS PARTS ?

If you are missing a part or if a part has been damaged, please contact dBb remond customer service to find out if parts are available and find out billing and shipping terms

- e-mail: contact@dbb-remond.fr
- phone: 01 49 15 96 40
- website: www.dbb-remond.fr

CONDENSATION OR MILK IN THE PIPES:

If you find condensation in the pipe, wash and dry immediately or replace the pipe. The dBb electronic breast pump has a non-return diaphragm on top of the pumping head to prevent milk from overflowing through the pipe into the breast pump motor block. If you see breast milk inside the pipes or in the end of the pipes, Do not try to wash, clean or sterilize the pipe or the end. Contact dBb Remond customer service instead.

TIPS

- if you feel discomfort or have pain while using the breast pump, try decrease the pumping level by turning the setting mode button of pumping.
- you can also try to collect your milk with or without the silicone cushion.
- if these tips do not help, turn off the breast pump and remove the device from your breasts for a few minutes. If you still feel pain or discomfort, consult a healthcare practitioner or a certified lactation consultant.

Position the device in order to easily disconnect the plug.

RADIO FREQUENCY

The breast pump requires special precautions regarding EMC and must be installed and put into service according to the EMC information provided in the user manual. Radio frequencies from cell phones and computers can affect the breast pump.

Note

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference from a residential installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and then back on, the user is encouraged to try to correct the interference by one or more of the following measures:

- reorient or relocate the device receiving interference.
- increase the distance between the equipment.
- connect the equipment to an outlet on a different circuit from the one where one or other devices are connected.
- consult the manufacturer or a service technician for help if you receive interference.

EMC/TECHNICAL DESCRIPTION

Guidance and manufacturer's declaration – electromagnetic emissions		
This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment.		
Emission tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The breast pump uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The breast pump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

Warning:

This breast pump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, this breast pump should be observed to verify normal operation in the configuration in which it will be used.

Guidance and manufacturer's declaration – electromagnetic immunity

This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment. The breast pump has anti-reflux as an essential performance, which is not influenced by electro-magnetic.

More details are described on page 4.

The electromagnetic interference immunity of this device has been tested and approved according to the following criteria:

1. Change of operation mode except reset into standby mode (pump is switched off) .
2. Change of pressure value more than 10%, when the device is operated as intended.
3. Change in programmable parameters, including mode and suction level.
4. Error indicated and is not restartable.
5. Permanent hardware failure is induced in the device.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4- 2	± 8 kV contact discharge ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air discharge	± 8 kV contact discharge ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air discharge	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.

EMC/TECHNICAL DESCRIPTION

Electrical fast transient/burst IEC 61000-4-4	± 2 kV, 100 kHz repetition frequency	± 0.5 kV, 100 kHz repetition frequency	z repetition frequency Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 0.5 kV, ± 1 kV Line-to-line	± 0.5 kV, ± 1 kV Line-to-line	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT;1 cycle At 0° 70% UT; 25 cycles At 0° 0% UT; 250 cycles	0 % UT ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT;1 cycle At 0° 70% UT; 25 cycles At 0° 0% UT; 250 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the breast pump requires continued operation during power mains interruptions, it is recommended that the breast pump be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m, 50 or 60 Hz	30 A/m, 50 Hz / 60 Hz	It may be necessary to position the breast pump further from sources of power frequency magnetic fields or to install magnetic shielding. The power frequency magnetic field should be measured in the intended installation location to assure that it is sufficiently low.

NOTE : UT is the a.c mains voltage prior to application of the test level.

EMC/TECHNICAL DESCRIPTION

Guidance and manufacturer's declaration - electromagnetic immunity			
This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4- 6	3V 0,15 MHz - 80 MHz z 6 V dans les bandes ISM et radio amateur entre 0,15 MHz z et 80 MHz 80% AM à 1 kHz	3V 0.15 MHz – 80 MHz z 6 V in ISM and amateur radio bands between 0.15 MHz z and 80 MHz 80% AM at 1 kHz	Field strengths from fixed RF transmitters, determined by an electromagnetic site survey; a. must be less than the compliance level in each frequency range. b. Interference may occur near equipment marked with the following symbol:
Radiated RF IEC 61000-4- 3	10 V/m (minimum) 80 MHz – 2.7 GHz z 80 % AM at 1 kHz	10 V/m (minimum) 80 MHz – 2.7 GHz z 80 % AM at 1 kHz	

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the breast pump is used exceeds the applicable RF compliance level above, the breast pump should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the breast pump.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m .

EMC/TECHNICAL DESCRIPTION

WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the breast pump including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Test specifications for enclosure port immunity to RF wireless communications device.

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	Immunity test level (V/m)
385	385 to 390	TETRA 400	Pulse modulation 18 Hz	1,8	0,3	27
450	430 to 470	GMRS 460, FRS 460	FM ± 5 KHZ DEVIATION 1KHZ SINE	2	0,3	28
710	704 to 787	LTE BAND 13, 17	Pulse modulation 18 HZ	0,2	0,3	9
745						
780						
810	800 to 960	GSM 800/900, TETRA 800, iDEN 820 CDMA 850, LTE Band 5	Pulse modulation 217 HZ	2	0,3	28
870						
930						
1720	1700 to 1990	GSM 1800, CDMA 1900, GSM 1900, DECT, LTE Band 1,2,3,25; UMTS	Pulse modulation 217 HZ	2	0,3	28
1845						
1970						
2450	2400 to 2570	Bluetooth WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 HZ	2	0,3	28
1720	5100 to 5800	WLAN 802.11 a/n	Pulse modulation 217 HZ	0,2	0,3	9
1845						
1970						

EMC tests have included the AC adapter.

WARRANTY

LIMITED WARRANTY ON CONSUMER PRODUCTS

The dBb Remond double electronic breast pump is guaranteed against any non-commercially acceptable material and manufacturing defect for the original consumer «buyer» who bought the new product from dBb Remond approved distributors or resellers. This warranty is valid for 2 years from the date of purchase of the breast pump and if it is used in accordance with the intended use as described in the information on the device and provided that all instructions operation and maintenance are strictly observed. Warranty coverage ends if you sell or lend the breast pump to another person.

Under this warranty, dBb Remond will repair or, at its discretion, replace the breast pump at no charge for these replacement parts or labor. This warranty does not apply if the device has been abused or altered. Only dBb Remond brand replacement parts are recommended for dBb Remond breastfeeding items.

TO THE EXTENT PERMITTED BY LAW, DBB REMOND CANNOT BE HELD RESPONSIBLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO, REPLACEMENT COSTS RESULTING FROM BREACH OF ANY WRITTEN OR IMPLIED WARRANTY.

If you wish to make a claim under this warranty, you will need the following:

- the original set of the dBb Remond breast pump.
- the receipt indicating the date of purchase.
- detailed description of the problem.

You, the buyer, are responsible for all shipping costs. DO NOT ship the unit without permission from dBb Remond or the distributor.

EXCLUSION OF WARRANTY

dBb Remond will not be responsible and the warranty will be canceled if the alleged defect or malfunction has been caused by:

- improper use of the device in violation of the instructions for use.
- incorrect or incorrectly carried out maintenance, modification or repair which is not authorized by dBb Remond.
- the use of non-dBb Remond brand spare parts.
- incorrect transport or shipment.

The warranty does not cover normal wear and tear on parts and accessories of the dBb Remond double electronic breast pump.

Power requirements:

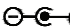
- AC / DC adapter (included):
 - > INPUT 100 V ~ 240 V AC, 50/60 Hz, 0.4 A max
 - > OUTPUT 9V DC / 1.3A
- Li-ion Battery (non-replaceable): 7.4 V 2200 mAh

Technical specifications :

- Vacuum: Soft pumping -75 ~ - 179mmHg (-10 ~ -24kpa)
Natural pumping -75 ~ - 285mmHg (-10 ~ -38kpa)
- ambient temperature range: + 10 ° C to 40 ° C (50 ° F to 104 ° F)
- relative humidity range: 20% to 80%
- transport / storage environment: -10 to 50 ° C / 20 to 90% relative humidity
- estimated Use-life: 1800 hours
- model: PA 209
- reference: 178515

SYMBOLS LEGEND

Symbols found on the packaging:	
	Dispose the device in accordance with local regulation.
	Type BF equipment (breastshield).
	Consult the instruction for use.
	CE mark (EMC/Electromagnetic compatibility).
	Product intended to come into contact with foodstuffs.

Symbols found on the device:	
	Indicates On/Off button.
	Dispose the device in accordance with local regulation.
	Type BF equipment (breastshield).
	Consult the instruction for use.
	Indicates direct current.
	Polarity of power supply.
	CE mark (EMC/Electromagnetic compatibility).

SYMBOLS LEGEND

Symbols found on the power adaptor:	
RoHS	Indicates the product compliance with the requirements of the Federal Communications Commission.
	Indicates the device for indoor use only.
	Dispose the device in accordance with local regulation.
	Indicates direct current.
	Polarity of power supply.
	CLASS II Equipment.
	CE mark (EMC/Electromagnetic compatibility).

Symbols found on the instructions manual:	
	Dispose the device in accordance with local regulation.
	Type BF equipment (breastshield).

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54 Rue Rouget de Lisle
93697 PANTIN CEDEX

+33 (0)1 49 15 96 40

www.dBb-remond.fr

contact@dbb-remond.fr

www.facebook.com/dBb.Remond

www.instagram.com/dbbremond